


HOW TO USE SEATRAC'S MOVER REMOTE CONTROL

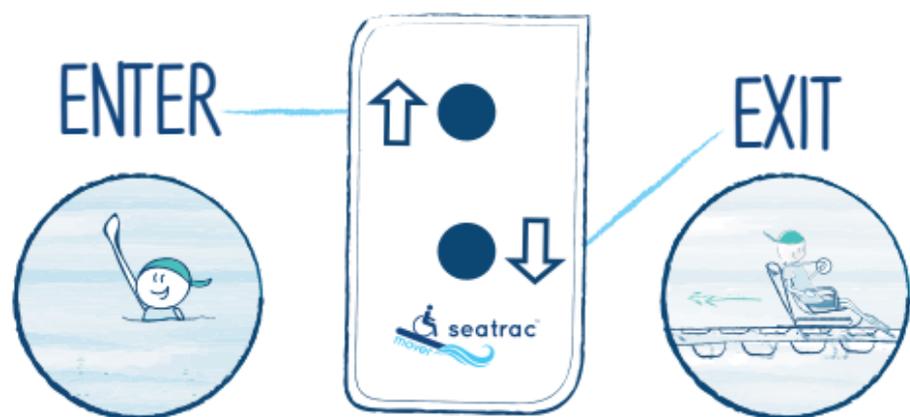

- ① In order to go into the water, keep the top button constantly pressed (ENTER)
- ② In order to return to the shore, keep the bottom button constantly pressed (EXIT)
- ③ In order to stop, I release any pressed button

* If it is in the waterproof cover, I can swim with it.
DO NOT remove the plastic cover.

I make a full ride,
in and out of the water
before embarking on
SEATRAC's chair.

Be sure to sanitize
the chair with a
disinfectant wipe
before use.

In case of
extreme waves,
I don't use
SEATRAC.

